



SIMPLY MORE EFFICIENT.

AFINION™ HbA1c

Consultations with patients who do not have their lab results are challenging and inefficient. Point-of-care HbA1c testing can improve operational efficiency and lead to increased patient motivation and satisfaction.¹⁻⁴

- 1.5 µL sample volume
- Laboratory-quality results in 3 minutes
- No interference from common Hb variants
- NGSP and IFCC certified



Afinion™ HbA1c

FACTS

The Afinion HbA1c is a simple fingerstick test that provides the patient HbA1c value within minutes, during the consultation.

- 1.5 µL sample volume
- Capillary and venous whole blood sample material
- Measuring range: 4-15% HbA1c/20-140 mmol/mol HbA1c
- 3 months room temperature storage
- Ready to use controls in 2 levels
- Integrated sampling device

HbA1c TESTING

The 3-minute Afinion HbA1c test fits perfectly within the workflow:

“Point-of-care HbA1c testing represents an opportunity for improved care of people with diabetes.”

—Egbinike V et. al. *Diabetes Educator*. 2013;39:66-73.

“Point-of-care testing for HbA1c provides the opportunity for more timely treatment changes.”

—ADA. *Diabetes Care* 2018;41(Suppl. 1):S55–S64.

3-STEP PROCEDURE

- 1 Collect the sample with the integrated sampling device.



- 2 Place the sampling device back in the test cartridge.



- 3 Place the test cartridge in the analyser and close the lid. The processing starts automatically.



IDF RECOMMENDATIONS

International Diabetes Federation (IDF) recommendations for use of HbA1c in type 2 diabetes⁵

- Diagnostic criteria: HbA1c \geq 6.5% or HbA1c \geq 48 mmol/mol
- Provide HbA1c result, measured at site of-care or in the laboratory, before the clinical consultation.
- Measure HbA1c every 2 to 6 months depending on level, stability of blood glucose control and changes in therapy.

AVAILABLE TESTS

AFINION™ ACR
AFINION™ CRP
AFINION™ HbA1c
AFINION™ LIPID PANEL

**CONTACT US TODAY FOR MORE INFORMATION OR TO ARRANGE A DEMONSTRATION
VISIT ABBOTT.COM/POCT**

T: 1800 622 642 (free call AU) | T: 0800 466 951 (free call NZ) | T: +61 7 3363 7100 | rapiddx.ANZ.enquiries@abbott.com | abbott.com/poct

1. Shepard MD. *Clin Biochem Rev*. 2006;27(3):161-70.
2. Laurence CO, Gialamas A, Bubner T. *Br J Gen Pract*. 2010;60(572):e98-e104.
3. Miller CD, Barnes CS, Phillips LS, et al. *Diabetes Care*. 2003;26(4):1158-63.
4. Crocker JB, Lee-Lewandrowski E, Lewandrowski N, et al. *Am J Clin Pathol*. 2014;142:640-6
5. Global Guideline for Type 2 Diabetes, IDF, 2012. ISBN 2-930229-43-8.

© 2019 Abbott. All Rights Reserved. Afinion is a trademark of the Abbott group of companies. All other trademarks referenced are trademarks of their respective owners. A2910.2

