

STEP 1



Prepare site as per local protocol and ensure skin is clean and dry before application.
Remove centre face of dressing.
Retain securement tapes on backing.

STEP 2



Remove backing to uncover adhesive side of dressing.

STEP 3



Position clear window over insertion site, positioning the cut-out to enable access to IV cannula.
Apply gentle pressure across the surface of the dressing, ensuring adherence.

STEP 4



Position non-woven tabs under cannula and press to adhere ensuring the hub is positioned above dressing.

STEP 5



Smooth edges of dressing whilst gently removing the paper frame.
Ensure thorough adhesion by smoothing down dressing from centre to outer edges.

STEP 6



Apply securement tape(s) and insertion date label as preferred and required by local protocol.

REMOVAL



To minimise potential damage to fragile skin, AsGUARD® Exceed I.V dressings should be eased off using the “stretch and release” technique or with an adhesive remover wipe.

INDICATIONS

AsGuard® Exceed I.V. Dressings are used to dress IV insertion sites and to provide primary fixation of intravenous access devices.

CONTRAINDICATIONS

Should not be used if allergies to any of its components is known.