



# Use of the HARTMANN Continance Range Molicare® Premium Elastic



## Colour coding system

The colour coding system allows for easy identification of the various absorption levels.

-  **Light Blue** for heavy incontinence
-  **Pink** for very heavy incontinence
-  **Violet** for severe incontinence
-  **Dark Blue** for very severe incontinence
-  **Grey** overnight product range


## Waist measurement

Correct sizing and fitting is essential for the wearers comfort and to reduce leakage.

The correct size can be determined by measuring the wearers waist.


### MoliCare Premium Elastic

- S = 70 – 90 cm
- M = 85 – 120 cm
- L = 115 – 145 cm
- XL = 140 – 175 cm

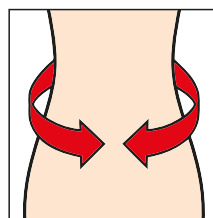

If the wearer is on the border between two sizes we recommend the smaller size.

## Wetness Indicator

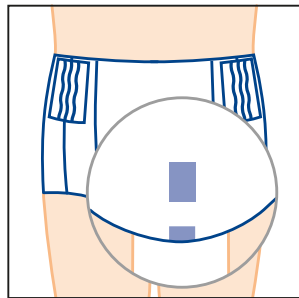


The unique curled fibre top layer in the HARTMANN continence range ensures that the wearer feels dry even after several voids. The pad does not need to be changed until it has reached 75% of full capacity.

The wetness indicator will smudge to indicate capacity and the pad should be changed when  $\frac{3}{4}$  of the wetness indicator has smudged/disappeared.

## Sizing

Sizing of elastic is indicated by easy to follow numbers of lines:

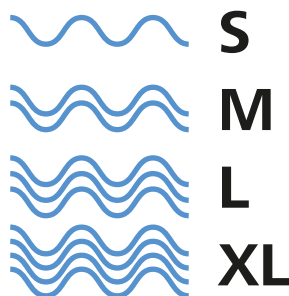

For further information on HARTMANN Continance and Skin Care products please contact your local Representative or call HARTMANN customer care toll free **1800 805 839**



[www.hartmann.com.au](http://www.hartmann.com.au)

PAUL HARTMANN Pty Ltd. ABN 35 000 099 589 Level 5, 1 Thomas Holt Drive  
Macquarie Park NSW 2113 PO Box 943, North Ryde BC NSW 1670  
© Registered trademark PAUL HARTMANN AG.