



STEP 1

The skin where application will take place should be clean and dry. As AsGUARD® Flex is a fixation dressing only, a dressing of choice should be used to cover any broken skin.

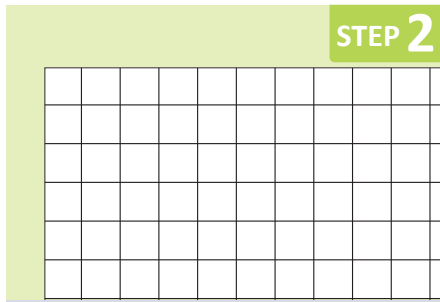


STEP 2

AsGUARD® Flex Non-woven size is dependent on the device being secured. Use grid to determine length – the Non-woven adhesive should extend 2-3cms beyond the device edges.

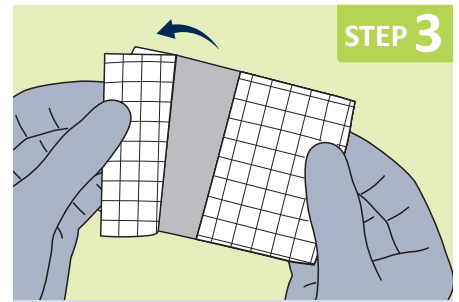


STEP 3

Cut to desired length. Remove backing paper from the AsGUARD® Flex.

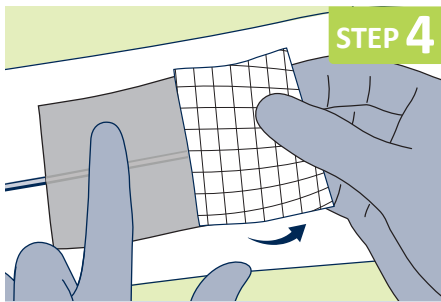

STEP 4


Apply the AsGUARD® Flex Non-woven dressing covering the device being secured. Release backing paper and apply over device.

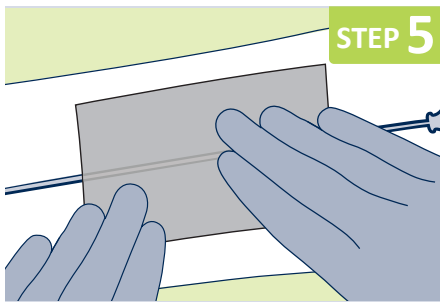

STEP 5

Smooth down the Non-woven adhesive from the centre to the edges, ensuring adherence.

DRESSING REMOVAL

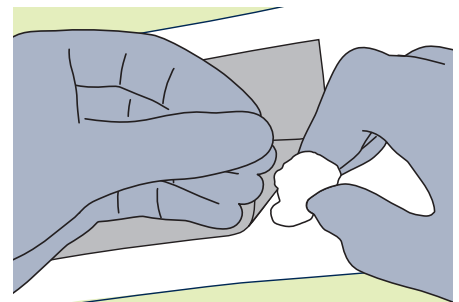

7 days is the maximum period AsGUARD® Flex should be left in situ.

To minimise potential damage to fragile skin, AsGUARD® Flex Non-woven Roll should be eased off using an adhesive remover wipe.

INDICATIONS

AsGUARD® Flex Non-woven Rolls are non sterile and indicated for use as a fixation layer to keep non-adhesive primary dressings and other medical devices in place.

CONTRAINDICATIONS

- Should not be used if allergies to any of its components, is known.

SMHO-AII-FU-0021_REVO_03_20 All product information is correct at date of publication