

## WHY SHOULD YOU TEST YOUR CIDEX® OPA SOLUTION?

To ensure each CIDEX® OPA Solution effectively destroys microorganisms, the minimum effective concentration (MEC) required for high-level disinfection should be tested with the appropriate CIDEX® OPA Solution Test Strip prior to each usage<sup>1</sup>.



**Completely submerge** the indicating pad of the strip into CIDEX® OPA Solution for one second and remove. Do not shake strip.



**Remove excess** solution by standing strip upright on a paper towel.



**Read results** at 90 seconds. Do not read past 90 seconds.

## ASP CIDEX® OPA Solution Test Strips

EASY ASSURANCE OF EFFICACY

CIDEX® OPA SOLUTION CAN BE USED OR REUSED FOR A MAXIMUM OF 14 DAYS.<sup>1</sup>

How to test: Completely submerge indicating pad of the strip into solution. Hold for 1 second and remove. Do not shake strip.

Remove excess solution by standing the strip upright on a paper towel.

Read results at 90 seconds. Do not read past 90 seconds. Pad will be completely PURPLE to indicate effective solution. IF ANY BLUE remains on the pad apart from the top line, solution is ineffective and must be discarded.

<sup>1</sup>CIDEX® OPA Solution may be used or reused for up to a maximum of 14 days provided the required condition of ortho-phthalaldehyde concentration and temperature exist based upon monitoring described in the Instructions for Use. Do not rely solely on days in use.

<sup>1</sup>Wear suitable protective gloves/protective clothing/eye protection/face protection.

**PASS**



**FAIL**



**FAIL**



Reorder #	Product description	Case contents
20392	CIDEX® OPA Solution Test Strips	2 bottles – 60 strips per bottle

**Protecting patients during their most critical moments.™**